



Hyperangulated blade technique

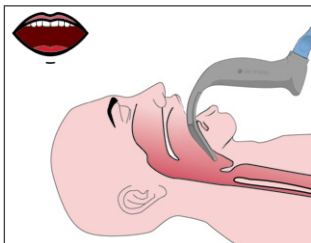


- 1** Insert the blade midline, advancing the tip until it is past the posterior aspect of the tongue.




- 2** Identify the epiglottis and advance the blade into the vallecula for a view of the glottis.

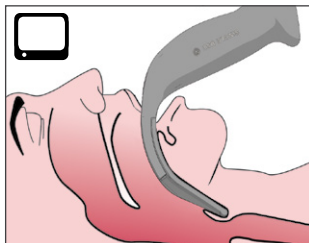


- 3** Using a GlideRite stylet (optional), introduce the ET tube along the right side the blade.

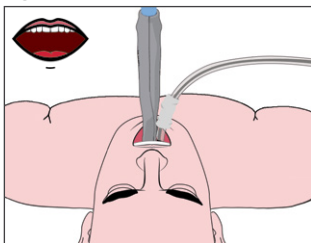


- 4** Complete the intubation.

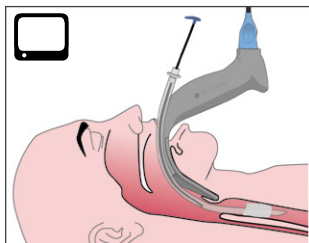


- 5** To remove the GlideRite stylet, pull it out towards the patient's feet.

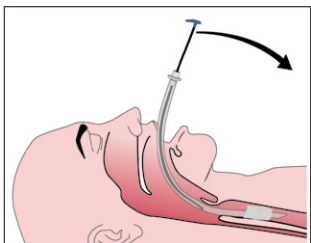

 — Look in the mouth during step.

 — Look at the monitor during step.

 *Important: this document is for quick reference only. For complete instructions, see your Operations & Maintenance Manual at verathon.com/service-and-support.*


 Verathon Medical (Canada) ULC
2227 Douglas Road
Burnaby, BC V5C 5A9, Canada
Tel: +1 604 439 3009
Fax: +1 604 439 3039

 Verathon Medical (Europe) B.V.
Willem Fenengastraat 13
1096 BL Amsterdam, The Netherlands
Tel: +31 (0) 20 210 30 91
Fax: +31 (0) 20 210 30 92



Verathon Inc.
20001 North Creek Parkway
Bothell, WA 98011, USA
Tel: +1 800 331 2313 (USA & Canada only)
Tel: +1 425 867 1348 Fax: +1 425 883 2896


 Anandic Medical Systems AG
Stadtweg 24
8245 Feuerthalen
Switzerland

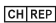 MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau
Switzerland

Verathon Medical (Australia) Pty Limited
Unit 9, 39 Herbert Street
St Leonards NSW 2065
Australia
Tel: 1800 613 603 / Fax: 1800 657 970 (within Australia)
Tel: +61 2 9431 2000 / Fax: +61 2 9475 1201 (International)

verathon

GlideRite, Verathon and their respective symbols are trademarks of Verathon Inc.
All other brand and product names are trademarks or registered trademarks of their respective owners.
© 2023 Verathon Inc.



0900-4511-ENUK REV-02