

FloPoint™ Digital Automatic Uroflowmeter

- *Helps diagnose abnormalities in voiding patterns*
- *Quickly assesses flow mechanics with automatic analysis*
- *Easy to use*
- *Economical*



The FloPoint™ Uroflowmeter records and charts a patient's pattern of urinary flow. The FloPoint™ system (Uroflowmeter and chair with load cell technology) helps physicians quickly assess the mechanics of flow and diagnose abnormalities in voiding patterns (e.g., obstructed or restricted flow, superflow, etc.).

Guided by the LCD display, the operator can enter patient information in seconds. The FloPoint™ Uroflowmeter features a high-performance microprocessor that compiles the enhanced urinary flow data at 8 samples per second, as the flow rate is displayed digitally. Advanced load cell technology provides superior accuracy, while minimizing maintenance and operating expense. The thermal printer charts the results. All of this helps streamline diagnostic uroflow (urine flow rate study) recording.

FloPoint™ Digital Automatic Uroflowmeter

HOW IT WORKS

```
Patient : .....
Number : .....
Facility : .....
Physician : .....
Sex : Female Age: 40 DOB: .....
Position : Sitting
Pattern : Continuous
Normal Void For Patient? Yes No
```

1. Enter patient info.

Patient header provides space for entering patient information, while printing age, sex, position, and Uroflow pattern (intermittent or continuous).


```
Q Max 22 ml/s
Q Avg 10 ml/s

T Flow 41 sec
T Void 41 sec
T to Max 02 sec
Vol 390 ml
Res.Vol .....
```


2. Perform Exam.

Sophisticated algorithms automatically analyze the Uroflow study.

3. FloPoint™ displays and prints exam results:

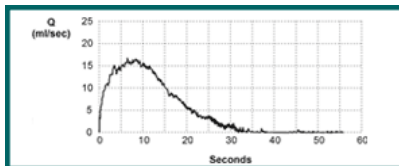

Normal Flow Pattern

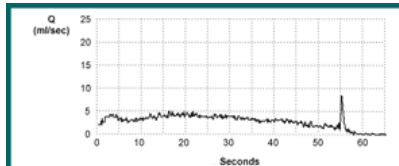

Bladder Outlet Obstruction



SPECIFICATIONS

Dimensions

FloPoint™ Uroflow/Printer:
5.5" x 7.5" x 3"
Load Cell: 5" x 3"
(12.5cm x 7.5cm)

Weight

FloPoint™ Uroflow/Printer:
1.8 lbs (820gm)
Load Cell: 1.4 lbs (640gm)

Includes

FloPoint™ load cell,
Uroflowmeter/printer,
chair*, 2 beakers, funnel,
2 rolls of printer paper

*Two sizes available

HOW TO ORDER

For more information about the FloPoint™ Uroflowmeter, please talk to your Verathon Medical™ representative or visit our website at www.verathon.com.



Corporate Headquarters

21222 30th Drive SE, Suite 120
Bothell, WA 98021
USA

(US and Canada only) 800.331.2313
425.867.1348

www.verathon.com

<https://my.scanpoint.com>