

Cleaning and Disinfecting the GlideScope® GVL and Cobalt Video Baton

- **DO NOT expose to temperatures above 60°C (140°F).**
Do not disinfect using devices such as autoclaves, ultrasonic cleaners or pasteurizers.
- The GlideScope® GVL is a non sterile, reusable device. It is recommended that the GlideScope® GVL is cleaned and disinfected after every patient use using a High Level Disinfectant method. High Level Disinfection is required for the GlideScope® GVL when it is visibly soiled.
- The GlideScope Cobalt Video Baton is a non sterile, reusable device. When used as intended, it is protected from direct contact with the patient by the sterile, single-use GVL® Stat. Low Level Disinfection is recommended for the video baton after every patient use. High Level Disinfection is required for the GlideScope® Cobalt Video Baton when it is visibly soiled.

1 Disconnect the GVL® or video baton from the monitor.

2 Place the cleaning cap over the connector as shown.

- During cleaning, the protective cap must be inserted as shown to protect the cable connector.

GVL®



Cobalt Video Baton



3 Wash the GVL® or video baton manually to remove all foreign material from the surface of the device.


- Chemical compatibility and disinfection methods are detailed in the User's Manual.
- To clean the exterior of the monitor and the video cables, wipe with IPA (70% isopropyl alcohol) bleach (100ppm) or a mild detergent and water.
- Wipe the cradle with a standard hospital-grade surface cleaning product.

For more detailed cleaning instructions see the GlideScope® System User's Manual.

Attaching/Detaching the GlideScope® GVL or Cobalt Video Baton

- 1 Insert the GVL® or Cobalt Video Baton cable into the port located on the face of the monitor so that the arrows on the cable and the monitor line up.
- 2 For GVL® only: Attach the opposite end of the video cable to the port located on the GVL®.

Note: When connecting and disconnecting the cable, grasp the connector by the gray sleeve.

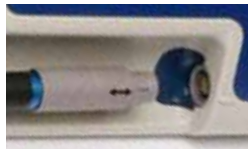

Attaching/Detaching the GlideScope® Cobalt Video Baton and Stat

- 1 Insert the Cobalt Video Baton into the sterile, single-use GVL® Stat until it clicks into place. Ensure proper insertion by matching the GlideScope® logo on the side of the video baton and the GVL® Stat.
- 2 Detach the video baton from the GVL® Stat by grasping the base of the video baton and pulling firmly.



IMPORTANT! A used GVL® Stat is a biohazard and should be disposed of in a manner consistent with local directives in the user's jurisdiction.

Corporate Headquarters:
Verathon Inc.
20001 North Creek Parkway
Bothell, WA 98011, USA
Tel: 800.331.2313 (US and Canada only)
Tel: 425.867.1348 Fax: 425.883.2896

Manufacturer:
Verathon Medical (Canada) ULC
4224 Manor Street
Burnaby, British Columbia
Canada, V5G 1B2

EC Representative:
Verathon Medical (Europe) B.V.
Linnaeusweg 11
3401 MS IJsselstein Netherlands
Tel: +31.30.68.70.570
Fax: +31.30.68.70.5121

VERATHON
MEDICAL
verathon.com