

GlideScope® System Quick Start

1 The monitor may be used immediately by plugging it into an AC power source, or by fully charging the battery prior to first use.

2 To ensure proper charging, follow these steps *in order:*

- Make sure the AC power cord is disconnected and the power switch is in the OFF position.
- Slide the power switch at the back of the monitor to the ON position.
- Plug the Monitor into an AC power source:
 - Insert the female end of the power cord into the port on the back of the Monitor.
- The charge status LED will turn orange, indicating that the recharging cycle has begun.
- When charging is complete, the status LED will turn green. At this point the unit is fully functional on battery power.



If the AC power cord is inserted before the power switch is in the ON position, the charge status LED will flash orange.

3 Connect the Video Laryngoscope to the monitor.

- If using the GVL®:
Connect either end of the video cable to the GVL® connector. Connect the video cable to the monitor connector.



GlideScope® System Quick Start




- If using the Cobalt system:

Connect the video baton to the monitor as shown for the GVL® above. Then slide a single-use GVL® Stat over the video baton.



To detach a GVL® Stat from a Cobalt Video Baton, grasp the base of the video baton and pull firmly.

IMPORTANT! A used GVL® Stat is a biohazard and should be disposed of in compliance with local protocols.

LED State	Meaning
Steady green  CHARGE STATUS	The battery is fully charged and ready for use.
Flashing orange  CHARGE STATUS	Flashing orange can indicate two states: <ul style="list-style-type: none"> If the AC power is connected and the power switch is off ("O" - to the left), the CHARGE STATUS LED will flash orange. The monitor will still function but the battery will not charge. If the AC power is NOT connected and the CHARGE STATUS LED flashes orange, the battery is malfunctioning.
Steady orange  CHARGE STATUS	Charging in progress

Corporate Headquarters:
Verathon Inc.
20001 North Creek Parkway
Bothell, WA 98011, USA
Tel: 800.331.2313 (US and Canada only)
Tel: 425.867.1348 Fax: 425.883.2896

Manufacturer:
Verathon Medical (Canada) ULC
4224 Manor Street
Burnaby, British Columbia
Canada, V5G 1B2

EC Representative:
Verathon Medical (Europe) B.V.
Linnaeusweg 11
3401 NS IJsselstein Netherlands
Tel: +31.30.68.70.570
Fax: +31.30.68.70.5121

 **VERATHON**
MEDICAL
www.verathon.com