


GlideScope® Ranger System Quick Start

1 The battery must be fully charged before first use. The unit will not operate while charging. If it is switched on during charging, the charging light will flash alternately green and orange.

2 To ensure proper charging, follow these steps *in order*:

- Make sure the power switch is in the OFF position.
- Unscrew the DC socket protective cap on the front of the monitor (a screwdriver or a small coin may be used).
- Plug the power supply barrel connector into the DC socket. Plug the AC cord into the power supply and then into the wall.
- The charge status LED will turn orange, indicating that the recharging cycle has begun.
- When charging is complete, the status LED will turn green.

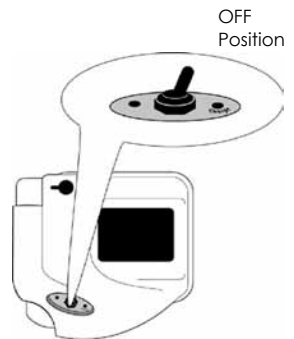


3 Connect the Video Laryngoscope to the monitor.

- If using the Ranger GVL®:

Insert the Ranger GVL® cable connector into the port located on the face of the monitor so that the arrows on the cable and the monitor line up.


- If using the Ranger Single Use system:

Connect the video baton cable connector to the monitor as shown for the Ranger GVL® above. Then slide a single-use GVL® Stat over the video baton.

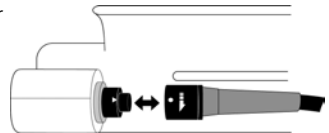



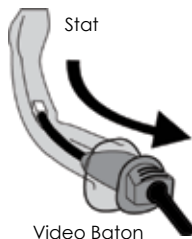

GlideScope® Ranger System Quick Start

To detach a GVL® Stat from a Ranger Video Baton, grasp the base of the video baton handle and pull firmly.



IMPORTANT! A used GVL® Stat is a biohazard and should be disposed of in compliance with local protocols.

LED State	Meaning
 CHARGE STATUS	The battery is fully charged.
 CHARGE STATUS	Battery power is low. LED flashes for approximately 5 minutes before shutting off.
 CHARGE STATUS	The battery is completely depleted and needs to be recharged.
 CHARGE STATUS	A flashing orange light indicates a battery problem. When this happens: <ul style="list-style-type: none"> • Disconnect the DC connector. • Make sure the switch is in the CHARGE (or OFF) position. • Reconnect the DC connector. • If the orange LED is still flashing, the battery has failed. • Return the unit for repair.

Corporate Headquarters:
Verathon Inc.
20001 North Creek Parkway
Bothell, WA 98011, USA
Tel: 800.331.2313 (US and Canada only)
Tel: 425.867.1348 Fax: 425.883.2896

Manufacturer:
Verathon Medical (Canada) ULC
4224 Manor Street
Burnaby, British Columbia
Canada, V5G 1B2

EC Representative:
Verathon Medical (Europe) B.V.
Linnaeusweg 11
3401 MS IJsselstein Netherlands
Tel: +31.30.68.70.570
Fax: +31.30.68.70.5121

 **VERATHON**
MEDICAL
www.verathon.com