

GlideRite® Auto Stylet

For Endotracheal Intubation

For use in 6.0 mm and larger endotracheal tubes.

The GlideRite® Auto Stylet was designed to help facilitate the placement of an endotracheal tube. The rigidity of this reusable stylet helps the user manipulate the tube as desired for intubation. The GlideRite Auto Stylet is adjustable to work with tubes of varying lengths (including cut tubes). It is optimized for use with GlideScope® Video Laryngoscopes.

WARNINGS


1. Product is NONSTERILE. Clean before use.
2. DO NOT use if the product appears damaged. INSPECT before use.
3. During use, the stylet should NOT protrude beyond the end of the endotracheal tube.
4. The stylet must not advance into the glottis under any circumstances.
5. DO NOT use the stylet for insertion past the vocal cords; the ventilation tube should be advanced off the stylet into the airway.

CLEANING

1. The product MUST be high-level disinfected before use.
2. The product must be cleaned of all visible contamination before disinfection.
3. Approved disinfection solutions are: .55% Ortho-Phthalaldehyde, 7.5% Hydrogen Peroxide and 3.4% Glutaraldehyde. Follow the chemical cleaner manufacturer's instructions for cleaning cycle parameters.
4. Disinfection with high-concentration chlorine solutions (>5000 ppm) may result in stainless steel rod corrosion.

STERILIZATION (for end user convenience)

If desired, the product can be sterilized using a minimum 4 minute 132°C (270°F) pre-vacuum steam sterilization cycle.

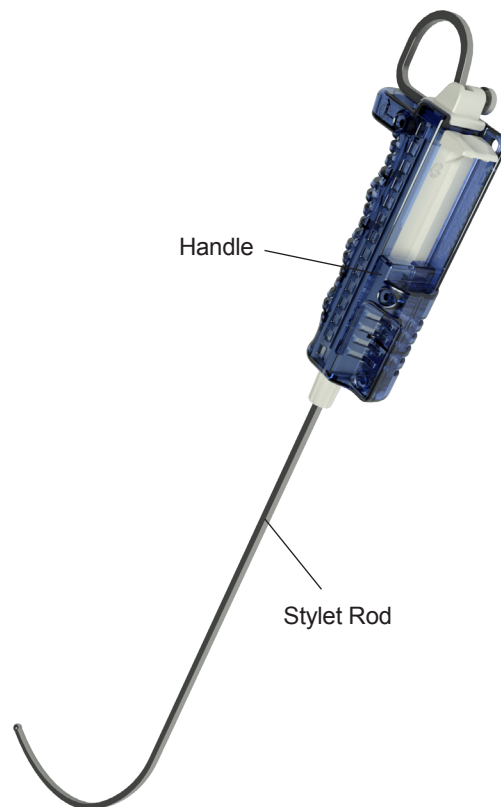

INSTRUCTIONS FOR USE:
Please refer to the annotated illustration on the back of this page.



**Read all instructions
prior to use**



Corporate Headquarters
20001 North Creek Parkway
Bothell, WA, 98011, USA
+425.867.1348 Fax: 425.883.2896

Verathon Medical (Europe) B.V.
Linnaeusweg 11
3401MS IJsselstein, The Netherlands
+31.30.68.70.570 Fax: +31.30.68.70.512

Verathon Medical (Canada) ULC
2227 Douglas Road, Burnaby, BC
V5C 5A9, Canada
+604.439.3009 Fax: 604.439.3039

GlideRite® Auto Stylet

For Endotracheal Intubation

INSTRUCTIONS FOR USE

Loading the ET Tube

1. Ensure **Thumb Screw** (B) is tightened to prevent the **Stylet Rod** (F) from moving.
2. While holding the **Thumb Activator** (D) in the up position (proximal end), slide the ET Tube over the **Stylet Rod** until the **ET Tube Connector** (E) on the **Handle** (C) engages the ET tube connector on the proximal end of the ET Tube.
3. If the distal tip of the **Stylet Rod** does not align with the distal tip of the ET tube, slightly loosen the **Thumb Screw** to allow the **Stylet Rod** to move freely through the **Handle**.
4. Position the **Stylet Rod** so the distal tips of the ET tube and Stylet are aligned. DO NOT extend the **Stylet Rod** beyond the distal tip of the ET tube.
5. Tighten the **Thumb Screw**.

Introducing the ET Tube

1. If the patient's neck or chest protrude to prevent introduction of the stylet and tube, consider introducing the stylet and tube at the 3 o'clock position (the right side of the mouth).
2. Position the distal tip of the ET Tube at the entrance to the vocal cords (DO NOT ADVANCE THE STYLET PAST THE VOCAL CORDS).
3. Depress the **Thumb Activator**. This will simultaneously advance the ET tube off the stylet and slightly past the vocal cords, while withdrawing the stylet a few centimeters.
4. Once proper tube position is confirmed, retract the stylet completely from the tube, pulling it toward the patient's feet.

**CLEAN AND DISINFECT THE
AUTO STYLET PRIOR TO REUSE.**

