

Cleaning and Disinfecting the GlideScope Ranger GVL and Ranger Single Use Video Baton


- **DO NOT expose to temperatures above 60°C (140°F).**
DO NOT disinfect using devices such as autoclaves, ultrasonic cleaners or pasteurizers.
- The GlideScope Ranger GVL® is a nonsterile, reusable device. It is required that the Ranger GVL is cleaned and disinfected after every patient use using a High Level Disinfectant method.
- The GlideScope Ranger video baton is a nonsterile, reusable device. When used as intended, it is protected from direct contact with the patient by the sterile, single-use GVL Stat. Low Level Disinfection is recommended for the Ranger video baton after every patient use. High Level Disinfection is required for the video baton when it is visibly soiled.

- 1 Disconnect the Ranger GVL or video baton from the monitor.
- 2 Place the cleaning cap over the connector as shown. Turn counter-clockwise to lock in place.
 - During cleaning, the protective cap must be inserted as shown to protect the cable connector.

Ranger GVL



Ranger Single Use video baton

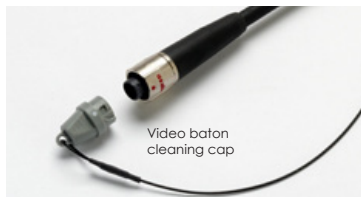


- 3 Wash the Ranger GVL or video baton manually to remove all foreign material from the surface of the device.
 - Chemical compatibility and disinfection methods are detailed in the user's manual.
 - To clean the exterior of the monitor, wipe with IPA (70% isopropyl alcohol) bleach (100ppm) or a mild detergent and water.

For more detailed cleaning instructions see the **GlideScope Ranger User's Manual**.


Attaching/Detaching the GlideScope Ranger GVL or Ranger Single Use Video Baton

- 1 Insert the Ranger GVL® or video baton cable into the port located on the face of the monitor so that the arrows on the cable and the monitor line up.
- 2 Detach the cable by turning the connector and gently pulling away from the monitor port.

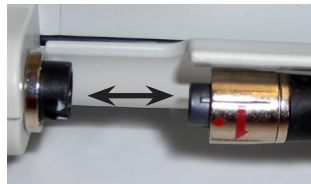

Attaching/Detaching the GlideScope Ranger Video Baton and Stat

- 1 To ensure proper insertion, match the GlideScope logo on the side of the video baton and the GVL Stat. Insert the Ranger video baton into the sterile, single-use GVL Stat until it clicks into place.
- 2 Detach the video baton from the GVL Stat by grasping the base of the video baton and pulling firmly.

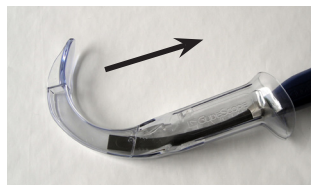

IMPORTANT! A used GVL Stat is a biohazard and should be disposed of in a manner consistent with local directives in the user's jurisdiction.

Corporate Headquarters:

Verathon Inc.
20001 North Creek Parkway
Bothell, WA 98011, USA
Tel: 800.331.2313 (US and Canada only)
Tel: 425.867.1348 Fax: 425.883.2896



Manufacturer:
Verathon Medical (Canada) ULC
2227 Douglas Road
Burnaby, British Columbia
Canada, V5C 5A9

ECTREP

EC Representative:
Verathon Medical (Europe) B.V.
Linnaeusweg 11
3401 MS IJsselstein Netherlands
Tel: +31.30.68.70.570
Fax: +31.30.68.70.5121

 **VERATHON**
MEDICAL
verathon.com