


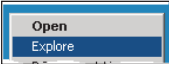

**For BladderScan® BVI 6100, BVI 6200, and BVI 6400**


Important! You must be a Local Administrator on this computer to install this program. If you are denied access, consult your supervisor or IT department.

**1 Place the ScanPoint® LC Software Install CD in your computer's CD-ROM Drive**

**2 InstallShield Wizard starts automatically**

If InstallShield Wizard does NOT open automatically:

- a. Use your mouse's RIGHT button to click **Start**. 
- b. Select **Explore** to open Windows Explorer®. 
- c. In the Explorer's menu tree on the left, scroll to select the CD drive.
- d. In the files displayed on the Explorer's right, double-click **setup**. 

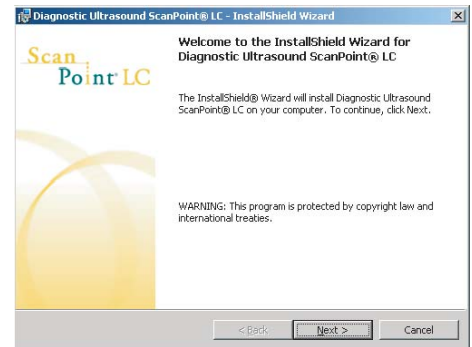


**3 Follow the Wizard's on-screen prompts**

- a. **License Agreement Window**  
You must accept the license terms to continue with the installation.
- b. **Destination Folder Window**  
Click **Next** to accept the folder settings commonly used to store files (default folders), OR click **Change** to select a different destination folder.

Location of ScanPoint® LC program files on your computer

Storage location for exam results you will be saving

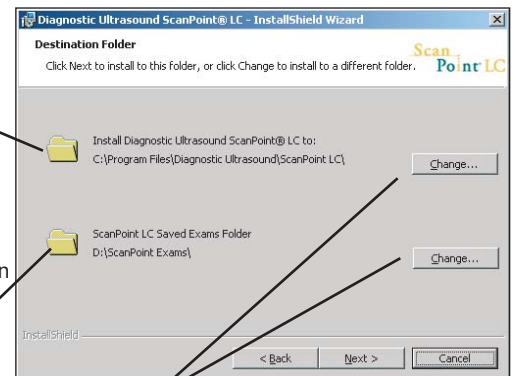


Optional: Select a different destination folder

**4 Finish InstallShield Wizard**

When you are satisfied with your settings, follow the InstallShield Wizard prompts to complete the installation.

NOTE: If the Program Maintenance window appears, then ScanPoint® LC was already installed on your computer. **Cancel** the installation and **Exit** InstallShield Wizard. If you have questions, consult your supervisor or IT department.

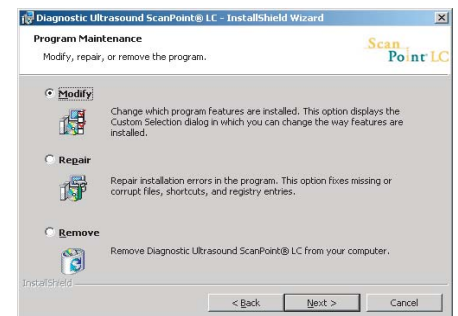

ScanPoint® LC



[www.verathon.com](http://www.verathon.com)

For more information, call 800.331.2313 (in the U.S. and Canada) or contact your local Verathon Medical® representative. BladderScan®, ScanPoint®, Sontac®, Verathon®, and Verathon Medical® are either registered trademarks or trademarks of Verathon Inc. in the United States and/or other countries. Copyright © 2006 by Verathon Inc. All rights reserved.

# SCANPOINT® LC QUICK START GUIDE


# ScanPoint® LC

## Save and View Exam Results for BladderScan® BVI 6100, BVI 6200, and BVI 6400

**1** Perform exam

**2** Place scanning unit in ScanPoint® Docking Station

**3** Open ScanPoint® LC

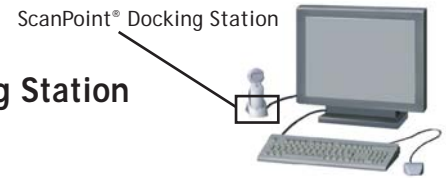


Double-click the ScanPoint LC icon on the Windows® Desktop.



Blank Exam Record Window:

ScanPoint® LC will display a blank Exam Record Window:

*Device is holding New Scan*  
OR  
*Device is holding Completed Scan*

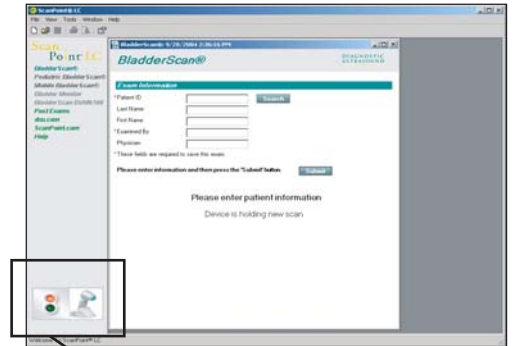

**4** Click **Submit** to transmit exam results to your computer

(Note: Mobile BladderScan® BVI 6400 voice-annotated exams will be transmitted automatically).

Status Box: Do not remove instrument from ScanPoint® Docking Station when red light is illuminated.

Do NOT remove scanning unit from ScanPoint® Docking Station until transmission is complete and green light appears in status box.

**5** View exam results on screen

- Bladder volume measurement (in ml)
- Six B-mode images presenting different views of the bladder



**6** Save exam results

- a. Enter Exam Information. (Note: Optional for Mobile BladderScan® voice-annotated exams)
- Patient ID (either type in or use Search)
  - Examined By (name of person who performed the exam)
  - Patient's Last Name and First Name (optional)
  - Name of Physician (optional)

b. Click **Save Exam** to open the Save As dialog box.


*Your exam will automatically appear using the format Patient ID\_date\_time. You can use this format or enter a different file name of your choosing.*

c. Click Save.

**7** To view saved exams

- a. Click Past Exams in the ScanPoint® LC main menu.
- b. To display a list of saved exams do one of the following:

*Enter Patient ID (type in or use Search), then click Go.*  
OR, click **Show All** to list all saved exams.

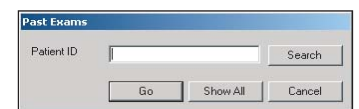

c. Use the check boxes to select one or more Exam IDs, then click Open Selected.



ScanPoint® LC



[www.verathon.com](http://www.verathon.com)

For more information, call 800.331.2313 (in the U.S. and Canada) or contact your local Verathon Medical® representative.

VM Corporate Headquarters: 20001 North Creek Parkway, Bothell, WA 98011-8218, USA. Phone: +1.425.867.1348

VM Europe B.V.: Linnaeusweg 11, 3401 MS, IJsselstein, The Netherlands. Phone: +31.30.68.70.570

VM U.K. Ltd.: The Granary Manor Farm Courtyard, Aston Sandford, Bucks, HP17 8JB, U.K. Phone: +44(0).1844.299.207

