
INTUBATION TECHNIQUE FOR A LOPRO OR GVL STAT

1 Insert the blade midline, advancing the tip until it is past the posterior portion of the tongue.

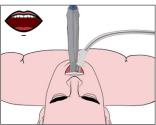


3 Introduce the ET tube along the right side the blade.






5 To remove the GlideRite stylet, pull it out towards the patient's feet.

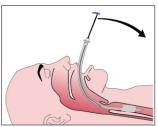



2 Identify the epiglottis and advance the blade into the vallecula for a view of the larynx.

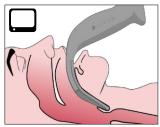



4 Complete the intubation.

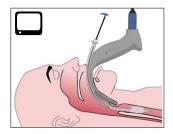

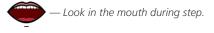

Look at the monitor during step.



Important: This document is for quick reference only. For complete instructions, see your Operations & Maintenance Manual at verathon.com/product-documentation.

Verathon Medical (Canada) ULC 2227 Douglas Road Burnaby, BC V5C 5A9 Canada Tel: +1 604 439 3009 Fax: +1 604 439 3039

EC REP

Verathon Medical (Europe) B.V. Willem Fenengastraat 13 1096 BL Amsterdam, The Netherlands Tel: +31 (0) 20 210 30 91 Fax: +31 (0) 20 210 30 92 Verathon Medical (Australia) Pty Limited Unit 9, 39 Herbert Street St Leonards NSW 2065 Australia Within Australia: 1800 613 603 Tel / 1800 657 970 Fax International: +61 2 9431 2000 Tel / +61 2 9475 1201 Fax



GlideRite, Verathon, and their respective symbols are trademarks of Verathon Inc. All other brand and product names are trademarks or registered trademarks of their respective owners. © 2021 Verathon Inc.