



INTUBATION TECHNIQUE FOR A LOPRO OR GVL STAT

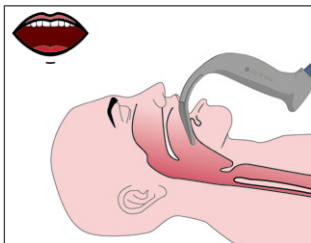


- 1** Insert the blade midline, advancing the tip until it is past the posterior portion of the tongue.




- 2** Identify the epiglottis and advance the blade into the vallecula for a view of the larynx.

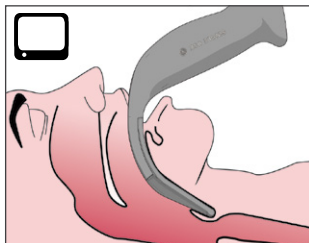


- 3** Introduce the ET tube along the right side the blade.

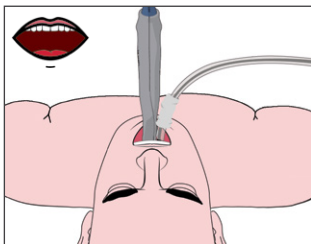

- 4** Complete the intubation.

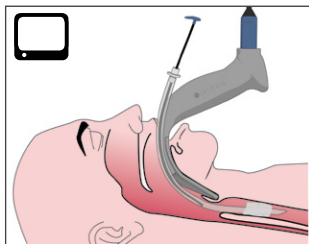


- 5** To remove the GlideRite stylet, pull it out towards the patient's feet.

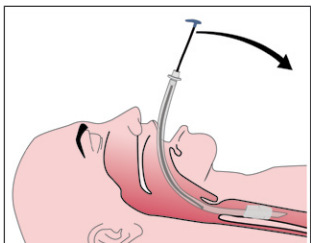

 — Look in the mouth during step.

 — Look at the monitor during step.

 *Important: This document is for quick reference only. For complete instructions, see your Operations & Maintenance Manual at verathon.com/product-documentation.*



Verathon Medical (Canada) ULC

2227 Douglas Road
Burnaby, BC V5C 5A9
Canada

Tel: +1 604 439 3009
Fax: +1 604 439 3039

EC REP

Verathon Medical (Europe) B.V.

Willem Fenengstraat 13
1096 BL Amsterdam,
The Netherlands

Tel: +31 (0) 20 210 30 91
Fax: +31 (0) 20 210 30 92

Verathon Medical (Australia) Pty Limited

Unit 9, 39 Herbert Street
St Leonards NSW 2065
Australia

Within Australia:
1800 613 603 Tel / 1800 657 970 Fax
International:
+61 2 9431 2000 Tel / +61 2 9475 1201 Fax



GlideRite, Verathon, and their respective symbols are trademarks of Verathon Inc.
All other brand and product names are trademarks or registered trademarks of their respective owners.
© 2021 Verathon Inc.



0900-4511-ENUK REV-01