
GlideRite®

verathon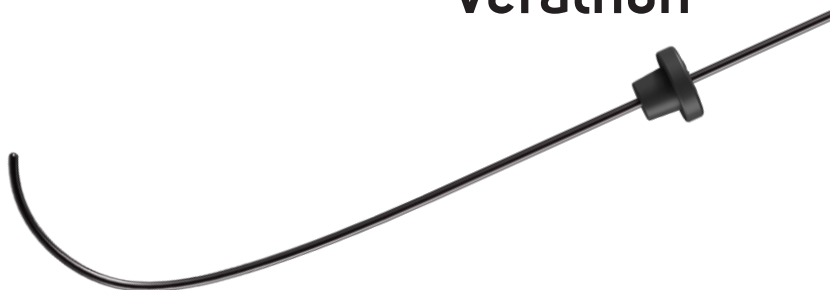

②

Single-Use Stylet Instructions for Use **EN**

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Product Description

The GlideRite® Single-Use Stylet (Small) was designed to help enable the placement of an endotracheal tube (also known as an *ETT* or *ET tube*). The rigidity of this stylet helps the user manipulate the tube as desired for intubation. The stylet is for use in ET tubes between 3.0 and 4.0 mm in internal diameter.

Note: There is no guarantee that instruments selected solely using these instrument dimensions will be compatible in combination.

Statement of Intended Use

The GlideRite Single-Use Stylet (Small) is an intubation-assist tool intended for use by qualified medical professionals, for guiding an endotracheal tube past the vocal cords into the trachea, shaped to work with GlideScope® video laryngoscope products.

Statement of Prescription

Caution: Federal (U.S.) law restricts this device to sale by or on the order of a physician.

WARNINGS

- Inspect before use—do not use if product or packaging is damaged.
- Do not allow the stylet to protrude beyond the distal end of the ET tube.
- Do not allow the stylet to advance into the glottis or past the vocal cords.
- Do not attempt to bend or reshape the stylet.
- Single use only. Do not reuse, reprocess, or resterilize.
- Dispose of the stylet as appropriate for biologically hazardous material.

Instructions for Use

1. Read all instructions before using the stylet.
2. Remove the stylet from its protective package.
3. Load a standard ET tube onto the stylet.
4. Slide the stylet into the tube until the distal end of the stylet is just inside the distal end of the tube.
5. Slide the stopper into position in the tube, so that the distal end of the stylet does not pass the distal end of the tube.

6. Position the distal end of the ET tube visually above the vocal cords. Do not advance the stylet past the vocal cords.
7. When the ET tube is engaged in the glottic opening, use your thumb to disengage the stopper and retract the stylet.
8. Position the ET tube per standard practice.
9. Remove the stylet from the ET tube.
10. Examine the stylet for damage, and then dispose of it as appropriate for biologically hazardous material.

Shipping Conditions

Temperature -20–45 °C (-4–113 °F) Humidity 10–95% Atmospheric Pressure 800–1060 hPa

Storage Conditions

Temperature 18–28°C (64–82°F) Humidity 40–60% Atmospheric Pressure 1013 hPa

Symbol Definitions

For a full glossary of symbols used by Verathon, visit [verathon.com/symbols](https://www.verathon.com/symbols).

California Residents — For Proposition 65 information, visit: [Verathon.com/Prop65](https://www.verathon.com/Prop65)

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© 2025 Verathon Inc.
GlideScope, GlideRite, Verathon, and associated symbols are trademarks of Verathon Inc.
Verathon Inc.
Pty Limited**
Not all Verathon Inc. products shown or described in this document are available for commercial sale in all countries.